

# Blendex<sup>®</sup>

*Automatic Alginate Mixer*



QMS

Blendex<sup>®</sup>

- Improved special gear design is more stable and durable with less noise and features built-in thermal overload protection.
- Automatic trouble-free alginate mixing at the touch of a button.
- Time saving process takes only seconds to complete.
- Consistent, smooth and bubble-free mixing each and every time.
- Features automatic timer with audible tone.
- 3 customizable memory presets available.
- LED display shows mixing time and memory presets.
- Automatic door safety lock ensures safe operation.
- Unit is available in AC100V 50/60Hz, AC110V 50/60Hz, or AC230V 50/60Hz operation.


**DENTAMERICA<sup>®</sup>**

# Blendex®

## Automatic Alginate Mixer



1 Add alginate and water to the mixing cup.

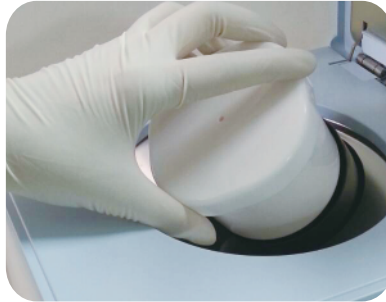


2 Place the mixing cup inside well.



3 Press the "START" button for a perfectly smooth Blendex® mix.


The Blendex system is equipped with 3 pre-set timing modes (these settings can be changed.)

Memory	Mixing Time (sec)
☺	8
☺☺	10
☺☺☺	12

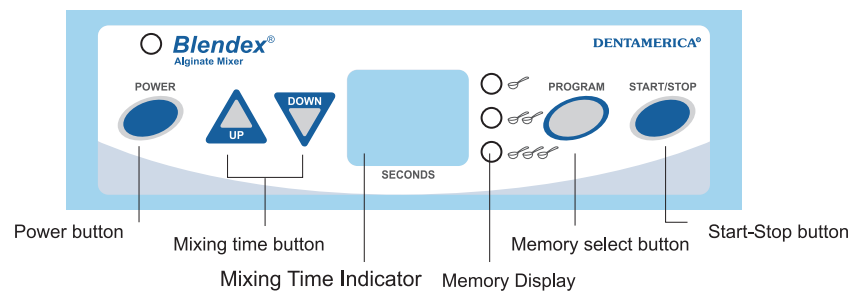

### SPECIFICATIONS

Dimension	W205 X D245 X H283 (mm)
Weight	17.5 kg (±0.5kg) N.W.
Voltage	AC230V (50/60 Hz) AC110V (50/60 Hz) AC100V (50/60 Hz)
Maximum Power	Maximum 350VA
Timer	1-99 sec, adjustable by time
Memory	3 (installed settings: 8, 10, 12 sec)
RPM	2,900 RPM

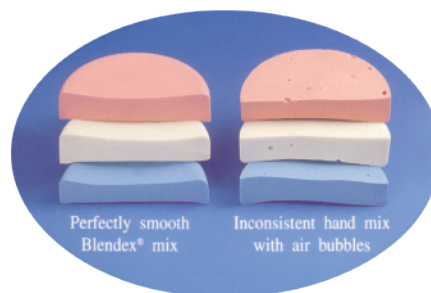

### DENTAMERICA®

18688 E. San Jose Ave.  
City of Industry, CA 91748 U.S.A.  
Phone: (626) 912-1388  
Fax: (626) 913-0510  
www.dentamerica.com  
www.dentamerica.asia

DISTRIBUTOR